

# nobi

## Welcome to Nobi

The Smartest Light in the World  
for Fall Prevention, Fall Detection & Smart Care



# Context


Why are we here?

nobi



Context

# Why are we here?

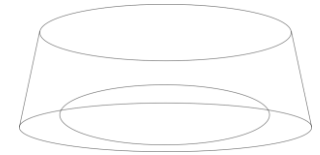

## Burlington Memory Care & Oak Harbor Memory Care

31 units & 25 units



1 year

### Internal reporting

- Nobi data: no
- Nobi intervention: no
- Nobi prevention: no

4 weeks – June / July 2024

### Nobi Learning

- Nobi data: yes
- Nobi intervention: no
- Nobi prevention: no

6 weeks – July / August 2024

### Nobi Active


- Nobi data: yes
- Nobi intervention: yes
- Nobi prevention: partial

Benchmark

Results

Context

# Why are we here?

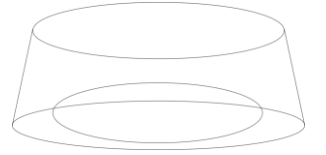

NOBI  
**Learning**

**Unwitnessed falls**  
per 1,000 resident days

4.2

34%

**Average intervention time**  
per fall



**Max intervention time**  
per fall



**Long lie > 30 min on floor**  
per 1,000 resident days

5.5

46%

**Very long lie > 1h on floor**  
per 1,000 resident days

4.1

34%

NOBI  
**Active**

0



**6**  
minutes




**15**  
minutes

0

0

Context

## Team 'Frontier Senior Living' & 'Nobi'

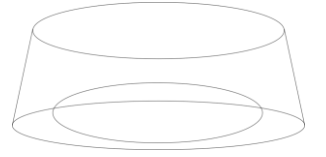

Yes, we can!



**Never again another  
unwitnessed fall**

Nobi supports you to  
ensure 100%  
detection of falls



**Hyper-Fast  
Intervention Time**

Together, we  
significantly reduce  
intervention time  
following a fall.



**Never again another  
long-lie fall**

Together, we ensure  
no resident is left  
waiting on the ground  
for hours.

The Result

**100%**  
peace-of-mind for  
residents, family & staff

---

**Time  
Saved**  
for our busy  
care staff




# Meet Nobi

nobi



Nobi  
The smartest light in the world

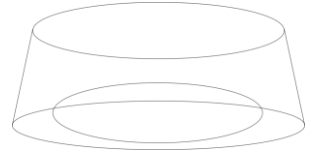

**Fall Detection**




**Fall Prevention**



**Smart Care**

# Nobi

## Nobi versus Nobita

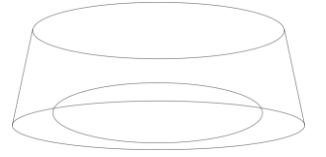

### Nobi



Nobi Ceiling

Nobi Pendant

- Bedroom & living room
- For high ceilings, we use the Nobi Pendant instead of the Nobi Ceiling.

### Nobita




- Designed for bathrooms, kitchens, and smaller rooms.
- Delivers the same functionality as Nobi.




Nobi

## Why Optical Sensors?

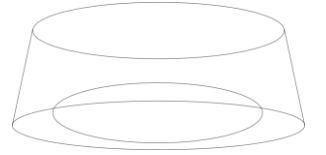

Compared to radar, **vision-based AI** is the most reliable technology for fall detection and prevention

- **Highly accurate fall detection**  
catches 100% of falls
- **Reduces false alarms**  
staff won't get overwhelmed by unnecessary alerts.
- **Live support during emergencies**  
During a fall, live (but anonymized) images can be streamed to help quickly assess the situation.
- **Fall Prevention Insights**  
Analyzing anonymous visuals of previous falls helps caregivers take steps to prevent future falls.

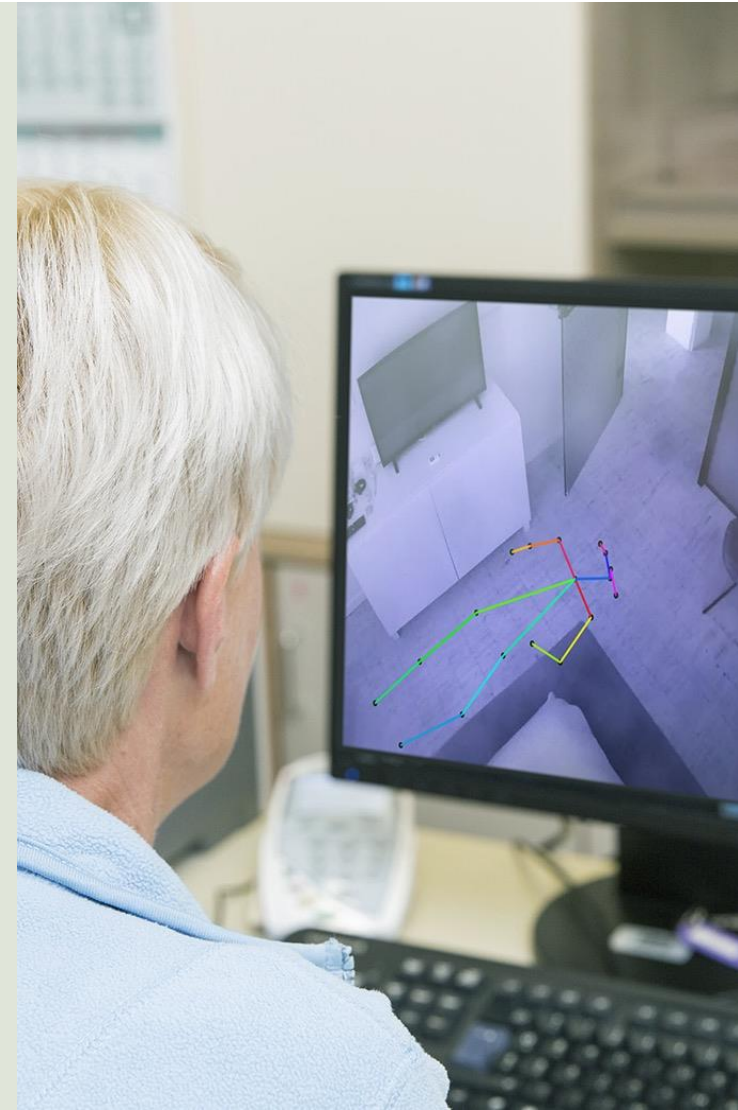

# How Nobi works


Fall Detection



nobi

# How Nobi works

## Fall Detection

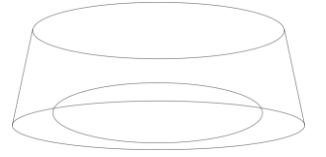

Nobi detects a fall



Alarm via phone call or nurse call system, and via dashboard



Immediate assistance from caregivers



Our loved ones are taken care of




Closing the escalation



# How Nobi Fall Detection works

## Different alerting channels

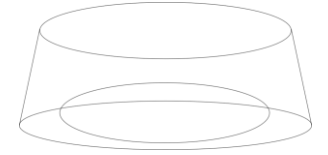


### Care institutions can choose how Nobi alerts caregivers after a fall or panic button press

In addition to notification on the Nobi Dashboard, the following options are available:



#### Phone

After a fall, Nobi calls/sends a message to care workers on their phones, and according to a cascade system.

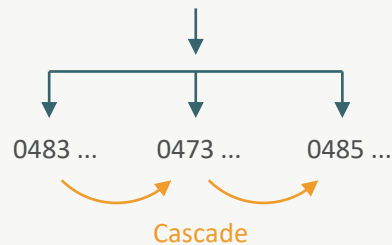


#### Nurse call system

The way a fall is reported depends on the respective nurse call system.




#### App

After a fall, employees also receive a push notification in the Nobi app. Notifications are sent simultaneously to all contacts.

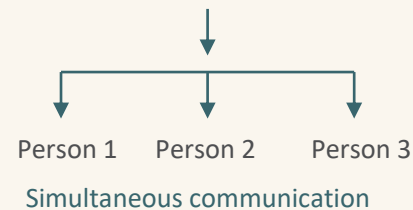

#### E-mail

After a fall, employees also receive an e-mail. E-mails are sent simultaneously to all contacts.

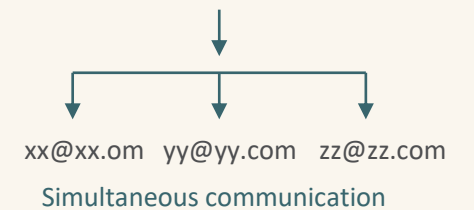


**Mandatory choice**

**Optional choice**

How Nobi Fall Detection works

Did you know?

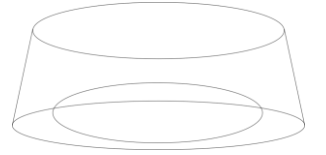

## 2. Nobi also detects slow falls

**Nobi also registers slow falls**  
as it is **not** a motion detection device

**So, unlike a smartwatch,**  
the falling speed doesn't matter





# How Nobi works


Fall Prevention



nobi

How Nobi works


# Fall Prevention

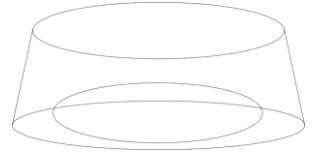

Automated  
lighting at night



Monitoring  
features

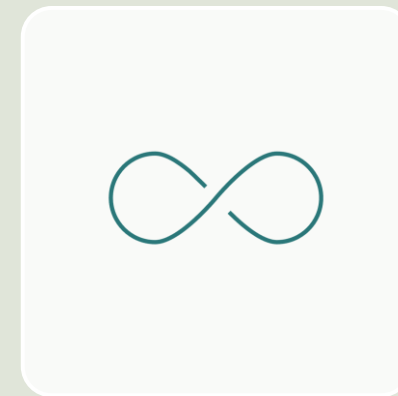


Fall  
analysis





How Nobi Fall Prevention works

# 1. Automated Night Light

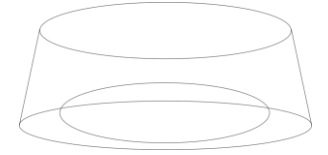


**When waking up during the night,  
good lighting is key in fall prevention**

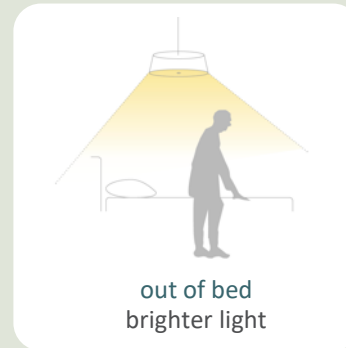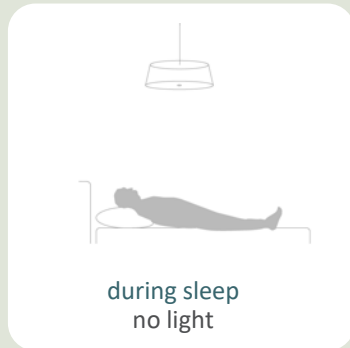

**Reduce  
disorientation**

**Less intrusive  
check-ins**


**Configurable  
per resident**

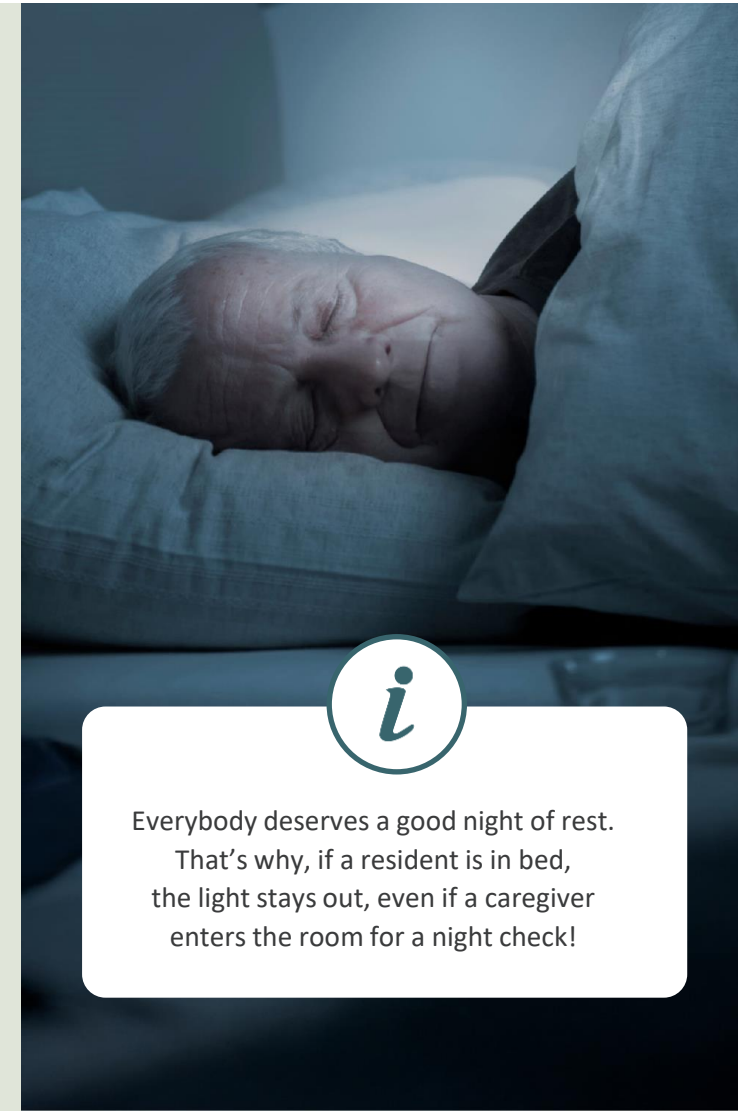

Everybody deserves a good night of rest.  
That's why, if a resident is in bed,  
the light stays out, even if a caregiver  
enters the room for a night check!

How Nobi Fall Prevention works

## 2. Monitoring functions

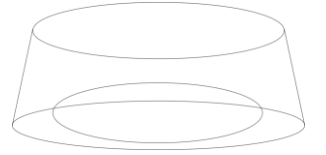

**When your resident is in a position with a high fall risk, Nobi notifies caregivers so they can proactively help**

These monitoring functions are available:



Resident is  
sitting on the bed  
(for xx minutes)



Resident is  
out of bed  
(for xx minutes)



Resident is  
in bathroom  
(for xx minutes)




No person  
detected



How Nobi Fall Prevention works

## 2. Monitoring functions

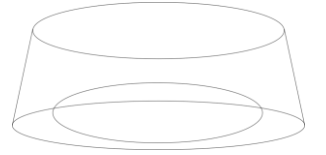

### Monitoring features can be customized for each resident

depending on their situation, medical condition

---

You can also configure:

When you want to monitor your resident

(nighttime, after 4PM, ...)

After how long you want to receive an alert


(2 minutes, 15 minutes, half an hour)







How Nobi Fall Prevention works

### 3. Fall analysis

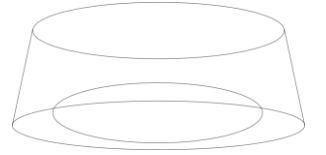

## Understanding the cause of a fall helps to prevent future falls

Nobi provides in-depth insights into falls by capturing the resident's behavior 15 seconds before and 15 seconds after the incident. With accompanying visuals and video, Nobi offers even more detailed context to better understand each fall.

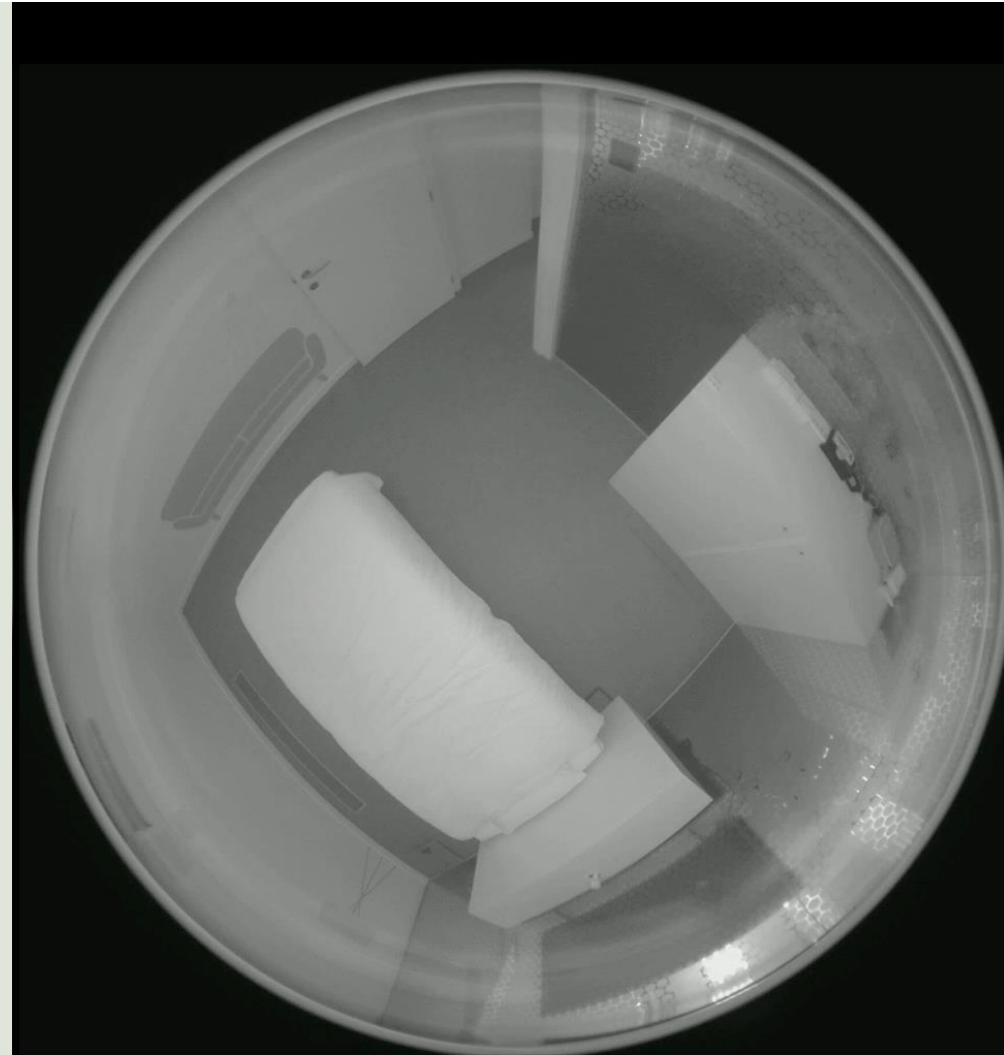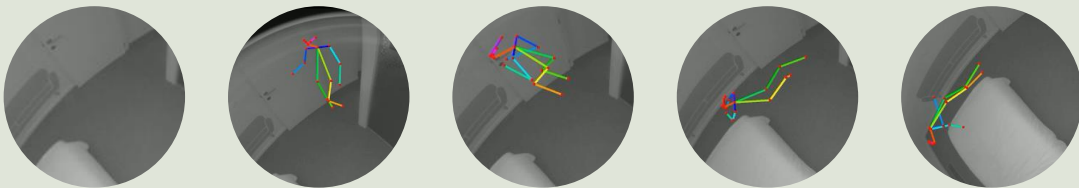


# How Nobi works

Smart Care



nobi

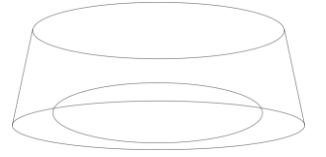


## Smarter care and more dignified lives

Nobi's feature of delivering insights help you to provide more individualized care and, in some cases, even to identify health issues at an early stage.




Live Check

+

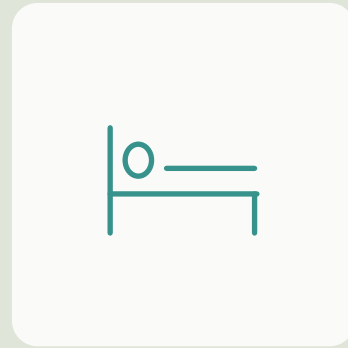

Sleep report

+

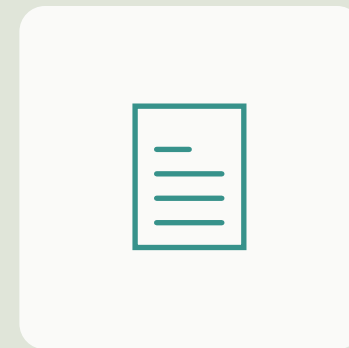


Escalation report  
for fall analysis



How Nobi Smart Care works

## 1. Live Check: dashboard

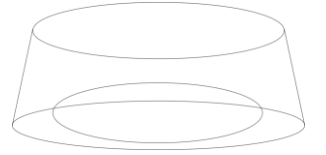

In Nobi's dashboard care staff can check the actual situation in each room, 24/7

With the resident's consent, you see:


Actual  
behavior

**Anonymized** stick figure  
on a fictional background


**Non-intrusive check-ins**

For example, when  
residents are sleeping

**More dignified living**

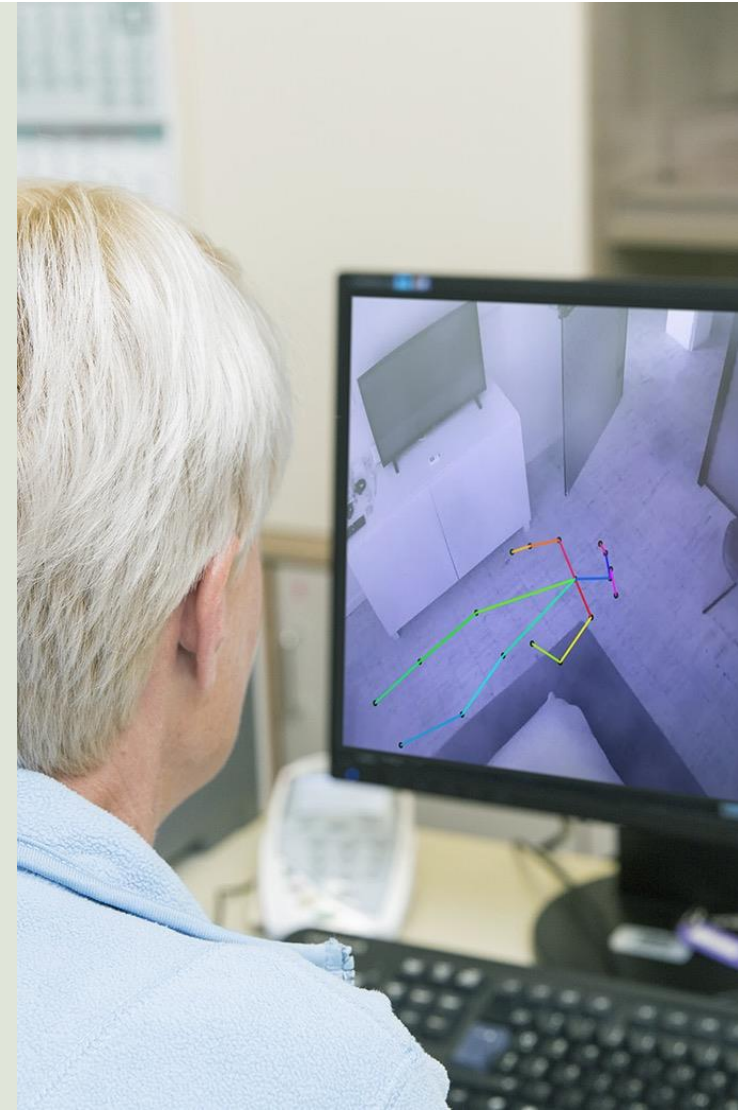

## 2. Night reports

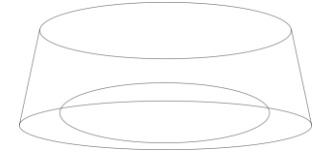

### Insights into in-bed and out-of-bed behavior



when the resident  
went to bed



how often the resident left  
the bed



total time spent in bed  
throughout the night

7 pm

■ in bed ■ out of bed

11 am


When caregivers notice a resident gets up three or four times each night and already is in bed at 7 PM, they can evaluate this behavior with the care team and search for solutions.





How Nobi Smart Care works

### 3. Fall analysis reports

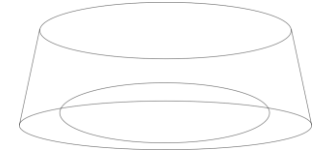


This report shows you an overview of the number of fall incidents.

#### REAL-LIFE SCENARIO - EXAMPLE

Care staff observed a significant number of falls occurring around 6 PM, coinciding with residents preparing for dinner.


After numerous slipping accidents, it seems that at 6pm it is dark in that part of the hallway. Extra light needs to be provided.

You can use this report over the next six months to assess whether there has been a reduction in falls.

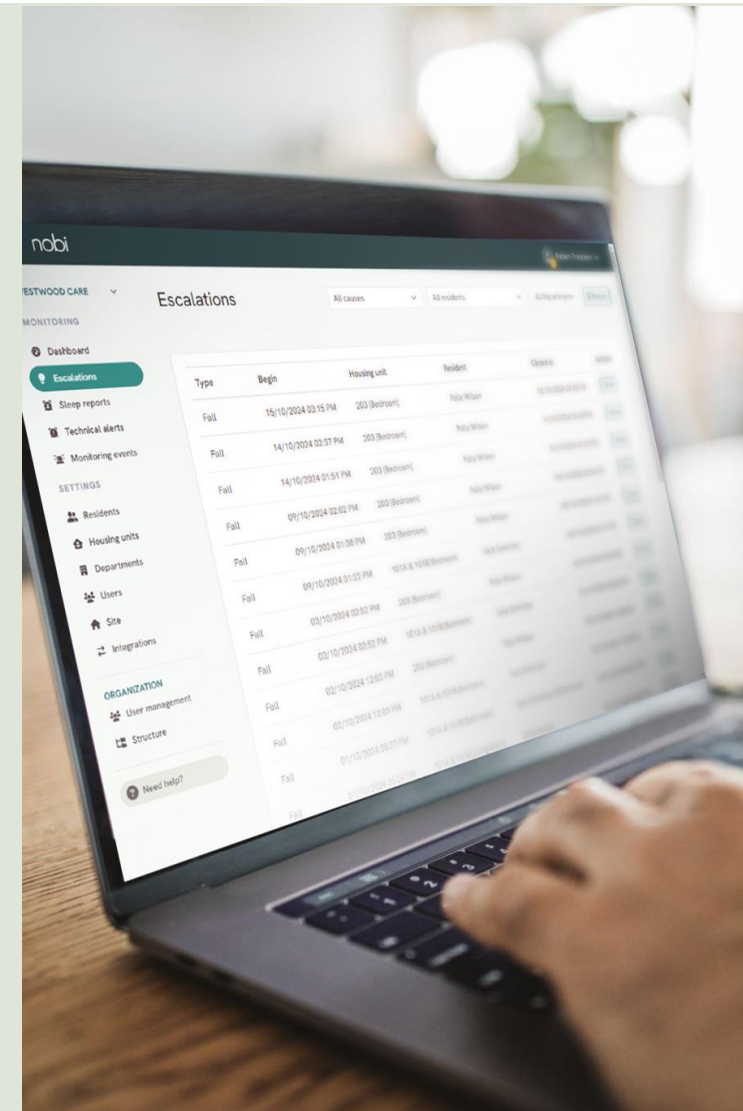

# How Nobi Smart Care works

## 3. Fall analysis reports

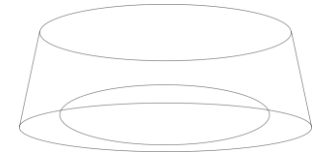

nobi Fabien Francken

WESTWOOD CARE

MONITORING

- Dashboard
- Escalations**
- Sleep reports
- Technical alerts
- Monitoring events

SETTINGS

- Residents
- Housing units
- Departments
- Users
- Site
- Integrations

ORGANIZATION


- User management
- Structure

Need help?

### Escalations

All causes All residents All Departments Export

Type	Begin	Housing unit	Resident	Closed at	Actions
Fall	15/10/2024 03:15 PM	203 (Bedroom)	Polly Wilson	15/10/2024 03:18 PM	Show
Fall	14/10/2024 02:37 PM	203 (Bedroom)	Polly Wilson	14/10/2024 02:44 PM	Show
Fall	14/10/2024 01:51 PM	203 (Bedroom)	Polly Wilson	14/10/2024 01:52 PM	Show
Fall	09/10/2024 02:02 PM	203 (Bedroom)	Polly Wilson	09/10/2024 02:03 PM	Show
Fall	09/10/2024 01:38 PM	203 (Bedroom)	Polly Wilson	09/10/2024 01:39 PM	Show
Fall	09/10/2024 01:22 PM	101A & 101B (Bedroom)	Jack Simmons	09/10/2024 01:27 PM	Show
Fall	03/10/2024 02:52 PM	203 (Bedroom)	Polly Wilson	03/10/2024 02:54 PM	Show
Fall	03/10/2024 02:52 PM	101A & 101B (Bedroom)	Jack Simmons	03/10/2024 03:05 PM	Show
Fall	02/10/2024 12:03 PM	203 (Bedroom)	Polly Wilson	02/10/2024 12:04 PM	Show
Fall	02/10/2024 12:03 PM	101A & 101B (Bedroom)	Jack Simmons	02/10/2024 12:04 PM	Show
Fall	01/10/2024 05:27 PM	101A & 101B (Bathroom)	Jack Simmons, Jill Simmons	01/10/2024 05:27 PM	Show

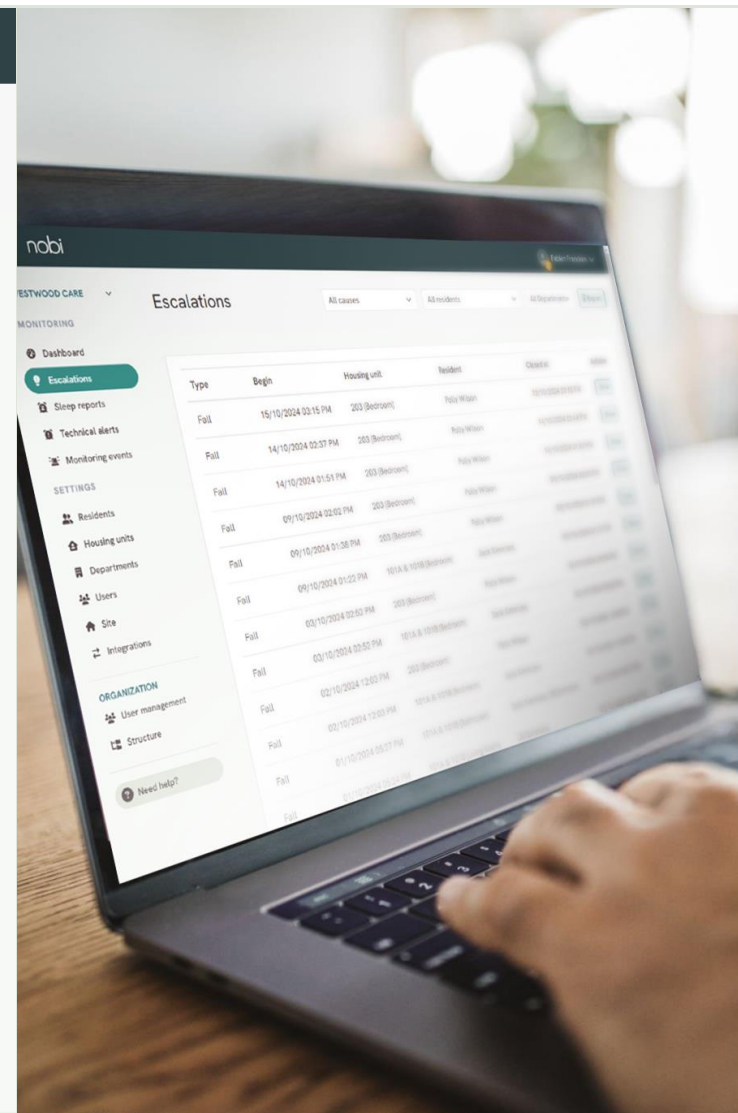

# 100% Privacy



for  
Residents


&

for  
Care Staff

nobi

100% privacy

# 1. No fall, no images sent to the cloud

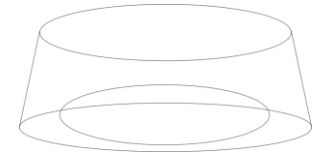

## Nobi chose for a strong Nvidia AI processor

that is located within the light itself

### The big advantage

**Interpretation of images happens locally:**  
No need to send images to the cloud for AI analysis.


**No internet dependency:**  
When linked with local call systems.

**No need for additional cabling:**  
Every light is processing its own data.



100% privacy

## 2. Residents are 100% in charge

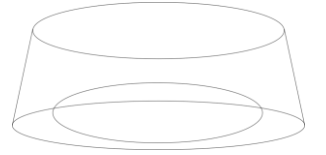

Before Nobi's smart features are activated, residents or their representatives choose what images can be viewed by care staff to gain more insights into the type of help needed after a fall and what caused the fall.

This way, Nobi helps care workers provide the right help after a fall and prevent future falls

### Privacy options




Full image



Anonymized stick figure



No image

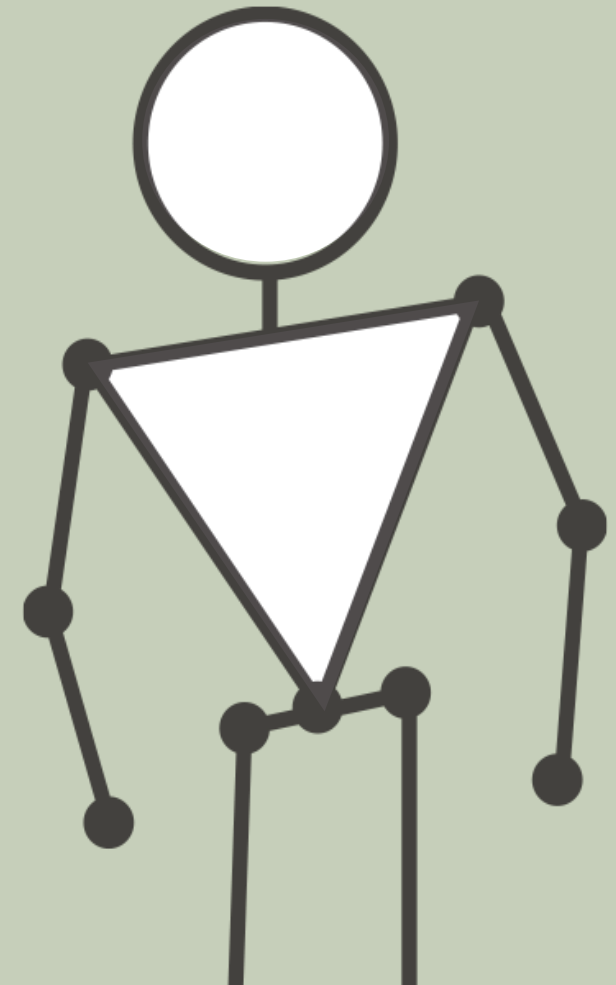




# What else is Nobi?

nobi



Nobi differentiators

## Meet the Nobi Dashboard

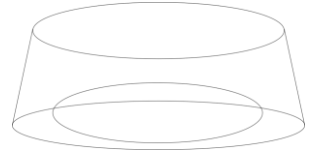

Displays residents requiring attention for caregivers at a glance:  
open escalation after a fall, monitoring event,  
a technical notification, ...

101A 101B

**Fall**

Bathroom at 01:53 PM (38 minutes ago)

Next steps


203 Fred Wilson

Attention Live view Sleep Parameters

**Out of bed**

Since 02:29 PM (2 minutes ago)


This monitoring event will disappear automatically once the situation is solved. (e.g. the resident is not "out of bed" anymore.)

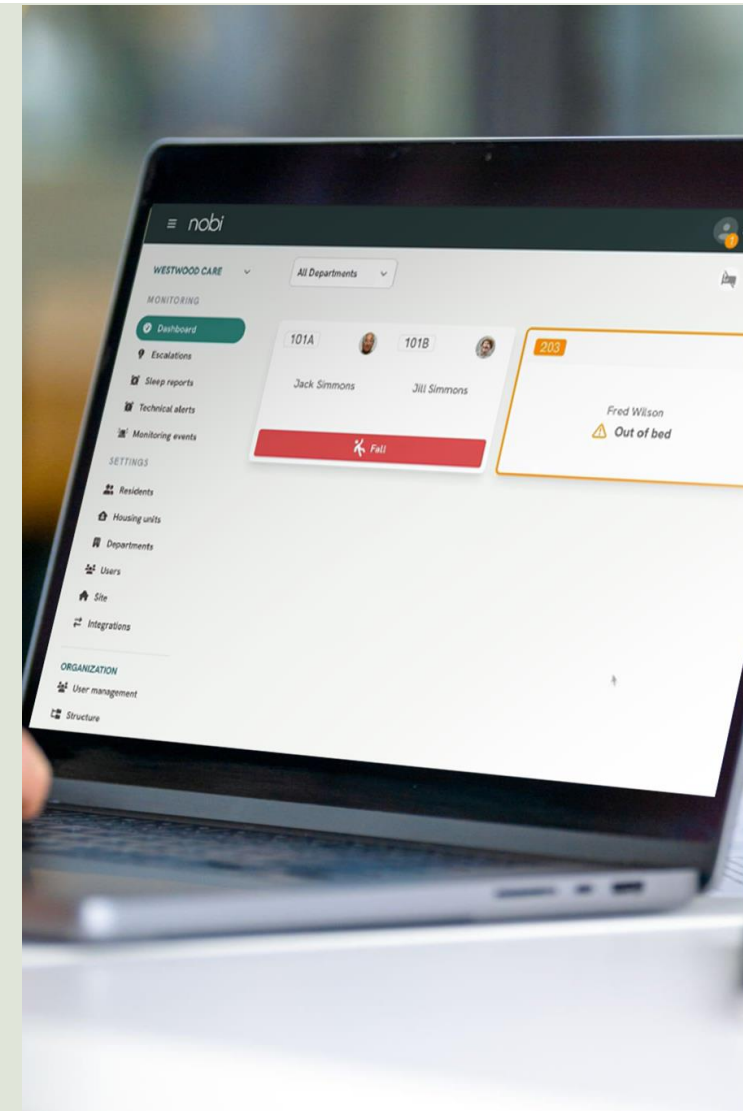



# Nobi differentiators


## Circadian rhythm

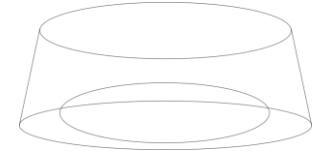

And then, there is also ... light

By following the natural body rhythm,  
Nobi's lighting plays a crucial role in the lives and well-being of residents.

Both brightness and color temperature adjust seamlessly to the time of day.

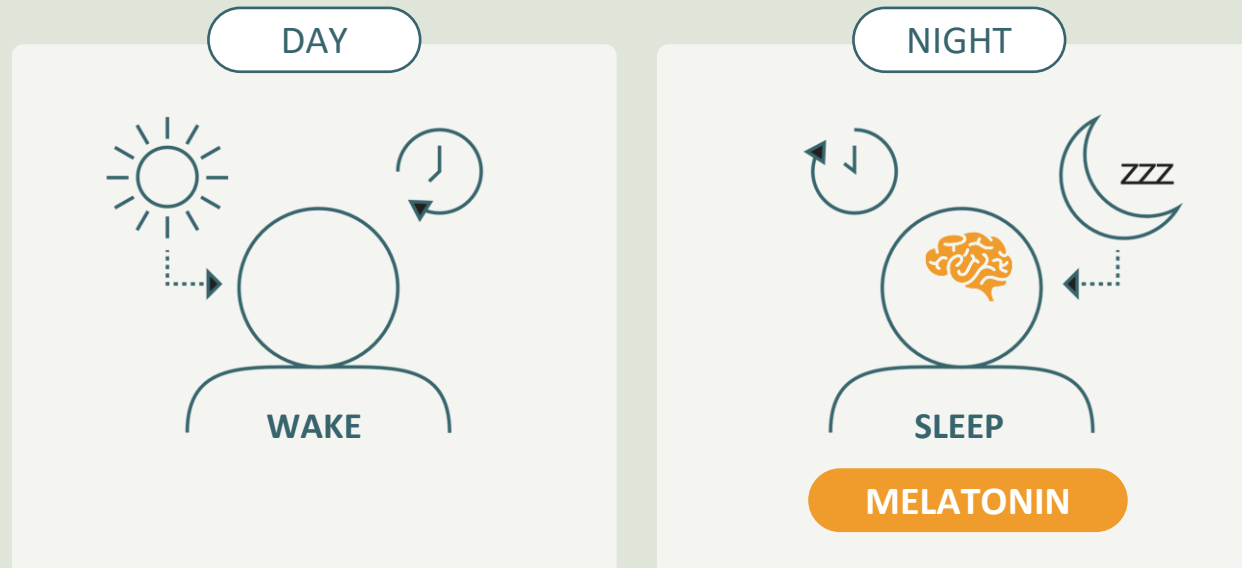



# What's next?

nobi



# What's Next?

## Nobi Onboarding Process

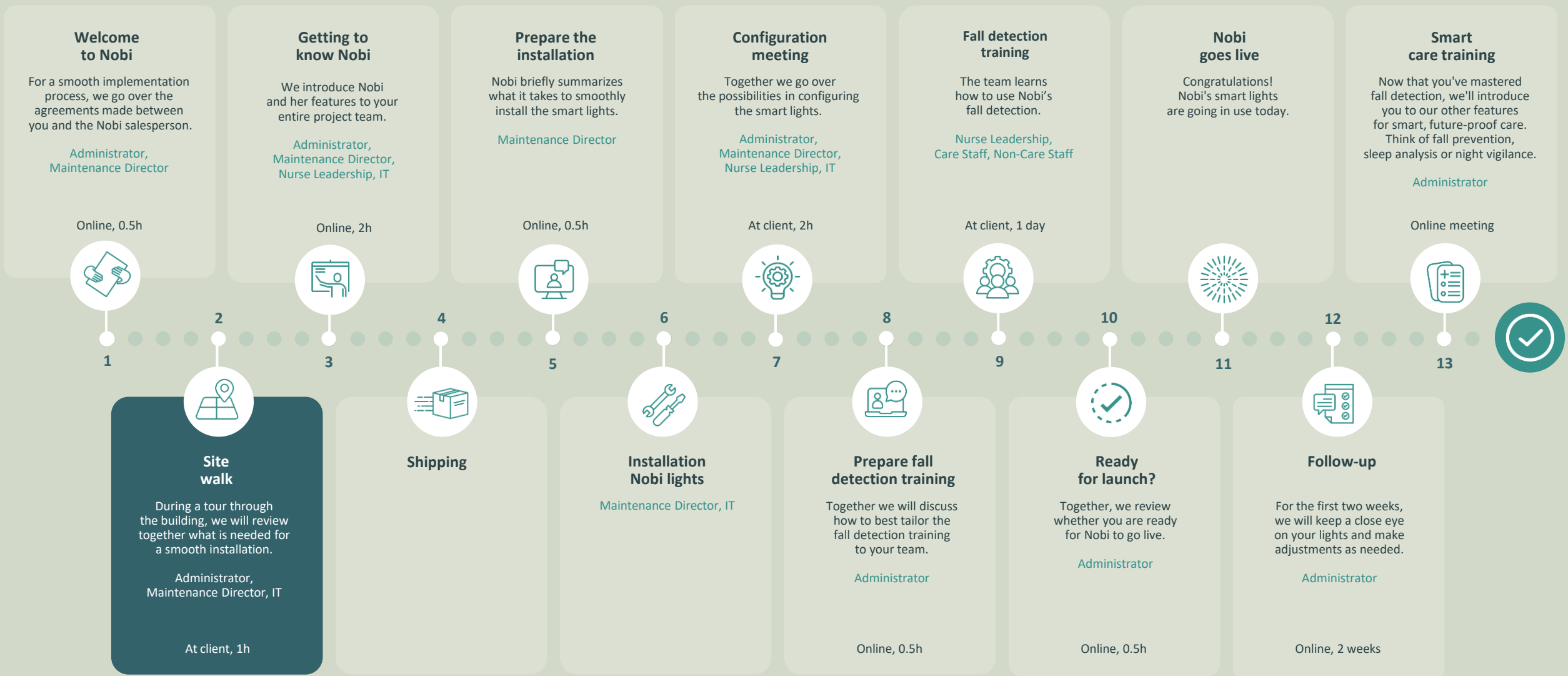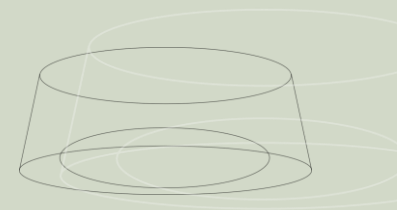

# Thanks

and take care

